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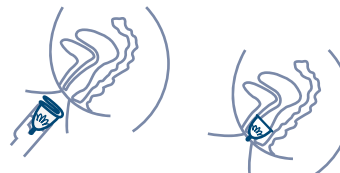
User instructions

How to use My Menstrual Cup

- 1 Before you use your cup for the first time, sterilise it in boiling water for at least 5 minutes.
- 2 Wash your hands and fold the cup, as in one of the pictures below.
- 3 Insert the folded cup carefully into your vagina, towards the base of your spine, without inserting it too far. A menstrual cup should be inserted lower than a tampon. The cup will open up inside your body, fitting perfectly to the walls of your vagina. Your vaginal muscles will hold the cup in place easily.
- 4 Check the cup is well positioned: pass your finger between the cup and the vaginal wall to check that it has unfolded correctly, if not rotate it slightly. The stem should be at the vagina entrance and accessible so you can remove it.
- 5 Remove the cup every 4-6 hours, depending on how heavy your flow is. Wash your hands, grip the stem then pinch the base of the cup (where the flower is). By squeezing, you will break the seal and the cup will fold so you can remove it.
- 6 Empty the cup and rinse it carefully with hot water. You can use a mild, neutral soap to clean it.
- 7 Sterilise your cup between cycles and keep it in its organic cotton pouch.

Tips:

- If you are using a cup for the first time, practise inserting it and removing it, perhaps at the end of a cycle. Don't worry, it'll take 2 or 3 cycles to get to grips with using your cup.
- If you are worried about leakage, you can use a sanitary towel as well as the cup, but this isn't necessary. Leakage is often a result of using a cup which is too small or positioning it too deeply in the vagina.
- If the stem of your menstrual cup is uncomfortable when you are sitting or walking, you can shorten it by cutting it. Be careful, however, not to cut the cup itself.
- If you don't have access to clean water when you are removing your cup (if you're using a public toilet for example), you can empty it and clean it with paper. Make sure you clean it carefully next time.
- Use only one internal sanitary protection at a time
- Use the menstrual cup only during the period.

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Choose your size:

Size: S – Volume: 16 ml – young women and light flows

Size: M – Volume: 21 ml – normal flows

Size: L – Volume: 28 ml – mums and heavy flows

Composition: 100% medical-grade silicone

Country of manufacture: France

Product lifetime: reusable for 10 years with careful maintenance

Distinctive feature: a flower at the base of the cup to make it easier to grip

For storage: a small organic cotton carrying pouch

Information and precautions

Toxic shock syndrome (TSS)

TSS is a rare but serious and sometimes fatal condition associated with the prolonged use of an internal protection product during menstruation. It is caused by a toxigenic strain of the *staphylococcus aureus* bacterium. This bacterium is naturally present in 20% of the population. It lives in the throat, nose, vagina and skin. You should be aware of its symptoms. The symptoms of TSS resemble those of the flu: sudden fever, vomiting, diarrhoea, dizziness or a sunburn-like rash during your period or on the days following your period. If you experience any of these symptoms while using the cup, remove it and consult your doctor immediately.

Do not use the cup if you have already experienced TSS in the past.

Do not wear the cup overnight. Do not wear the cup for more than 6 hours.

Thank you for choosing Anaë® and My Menstrual Cup!

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